

Mask of category UNS1 Made of reusable fabric 50 washes



INSTRUCTIONS FOR USE

WARNING

To protect your health and that of others, it is very important to follow these instructions for use.

- Wear this mask when you are in contact with people other than those you live with.
- Always check that the mask fits snugly and covers your mouth and nose.
- Warning: If you are ill, this mask may not fit. Ask your doctor for advice.
- This mask is not for use by nursing staff.
- This mask does not replace barrier gestures (regular hand washing, physical distancing, reducing contact with other people). It adds a physical barrier, to be used in particular when you are in close contact with other people.

How to wear the mask?

Before putting on:

- 01. Before touching the mask, wash your hands with soap and water or a hydro-alcoholic solution.
- 02. Inspect the mask and make sure that there is no holes, tears or damage.
- 03. It is recommended to wear the mask on bare skin, avoiding contact with the hair.
- 04. Never modify the mask in any way whatever it is.

To put it on:

- 01. Hold the mask by the elastic straps.
- 02. Place it at the level of your mouth.
- 03. Put the elastic straps behind your ears.
- 04. Adjust the mask so that it covers your nose, mouth and chin: you can pinch the nose clip to help you.

When you wear it:

- 01. Avoid touching and moving it.
- 02. Never put it in a waiting position on the forehead or on the chin.

It is necessary to change the mask:

- 01. When you wore the mask for 4 hours.
- 02. When you wish to eat or drink.
- 03. When it becomes difficult to breathe.
- 04. If the mask gets wet.
- 05. If the mask is damaged.
- 06. If the mask is deformed and no longer fits properly against your face.

To remove it:

- 01. Wash your hands with soap and water or hydro-alcoholic solution.
- 02. Unhook the elastic straps to take off the mask of your face.
- 03. While waiting to wash it, isolate it in a plastic bag.
- 04. If the mask is deformed or worn out, throw it away immediately in a closable bin.
- 05. To finish: wash your hands again with a soft cloth. water and soap or hydro-alcoholic solution.











How to take care of the mask

Before opening:

01. Store the mask in its original protective packaging, in a dry place and away from direct sunlight. of the sun. Store in maximum humidity of 75% and at a temperature between -2°C and + 55°C in order to preserve all its characteristics.

For washing:

- 01. Machine wash the mask with detergent. Choose a washing programme that includes at least 30 min at 60°C minimum. You can put it in the washing machine with your laundry.
- 02. After touching the mask, wash your hands with water and soap or hydroalcoholic solution.
- 03. This mask has been tested for 50 washes. After 50 washes, the filtration of the mask is no longer guaranteed.
- 04. When the machine is finished, take the mask out of the washing machine, and shape it again by flattening the barrette Nasal and by reshaping the folds without forcing.

After washing:

- 01. Dry the entire mask (the inner layers must also be completely dry)
- 02. For this you can: use a tumble dryer (making sure to clean the filters of the tumble dryer and wash your hands afterwards), use a hair dryer, or air dry your mask on a clean and disinfected surface.
- 03. Steam iron your mask at the temperature 130°C.
- 04. As soon as the mask is dry, store it in a clean, airtight package (plastic bag or freezer bag, for example).

Safety

- 01. This mask has demonstrated a filtration efficiency higher than 90% (99% when new and 97% after 50 washes) for 3 micron particles emitted by the person wearing the mask.
- 02. The respirability of the mask has been measured at > 96 L.m-2.s-1 (220 L.m².s-1 when new and 187 L.m².s-1 after 50 washes) and must allow it to be worn for a period of 4 hours.

Warning

Despite all the care taken in the manufacture of these masks, it is the sole responsibility of the user, and under no circumstances of the manufacturer, to ensure that the mask is suitable for the characteristics of the environment in which it is to be used. The recommendations and conditions of use described above must be scrupulously followed.

Counting of mask use

The masks can be washed 50 times, so each mask can be used 51 times. To help you monitor and manage the usage of your masks, please fill in the following table by ticking the boxes:

	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25
Mask 1																									
Mask 2																									
Mask 3																									
Mask 4																									

	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	49	50	51
Mask 1																										
Mask 2																										
Mask 3																										
Mask 4																										